FR

COMPOSITION

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PROPERTIES

So far, disinfection of handpieces, contra-angles, turbines and scaling inserts was very problematic.

Indeed, the use of surface sprays or wiping with disinfectant gauze squares could not ensure a disinfecting activity on all the germs.

Today, DECIDENT is the only system enabling to leave the instrument in contact with the disinfecting solution. In 15 minutes only the instrument is disinfected.

INDICATIONS

Disinfection of handpieces, contra-angles, turbines and inserts.

INSTRUCTIONS FOR USE

1. Clean instrument thoroughly with water to remove all debris (saliva, blood or other organic fluid).
2. Wipe the instrument dry.
3. Grasp tabs at upper corner of the sleeve and open.
4. Insert instrument completely into sleeve between the disinfectant-soaked sponges. Massage sleeve from top to bottom to distribute disinfectant thoroughly over the instrument surface and to provide a scrubbing action to ensure good surface contact with the disinfectant.
5. Leave instrument inside sleeve for at least 15 minutes.
6. Rinse instrument in running water and wipe it dry after removing it from the sleeve.

WARNINGS AND PRECAUTIONS FOR USE

Each pouch is designed to be disposable. Reuse results in critical risks to the safety of the user and the patient because of the possible deterioration of the solution, possibly leading to a loss of efficiency.

Highly flammable.

Keep away from sources of ignition. No smoking.

Keep container in a well-ventilated place.

Do not use on materials that could be damaged by alcohol.

STORAGE

Store below 25°C in a dry place and away from light.

MICROBIOLOGICAL PROPERTIES

Bactericidal in 15 min : EN 14561.

Mycobactericidal in 15 min : EN 14563 (Mycobacterium terrae, Mycobacterium avium).

Fungicidal in 15 min : EN 14562.

Active on HBV (B-Hepatitis), HIV (AIDS) and Herpes type 1 viruses in 15 min.

PRESENTATION

Box containing 24 disposable sleeves.

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